

Ways to use a Logan ProxTalker

The Logan ProxTalker communication device - empowering teachers and parents to support children's communication in different ways



Traditional personal requests and communication

Say words one at a time or use the green button to repeat the whole request.



Visual schedule

Turn ProxTalker on its end and set up schedule using tags identified as student needs. Easy to change for different time frames.



Story telling interaction

It's easy to generate story tags to allow students to participate. On Monday he ate "an apple".



Sequencing task

Create a set of verbal instructions for user to follow e.g. making scrambled egg, getting dressed...



Input to maths lesson

Use tags from Educational Set to let student participate in lesson. May aid math comprehension at the same time.



Real object schedule

Use real objects with tags to support students with visual or cognitive difficulties.

Commenting

Use the ProxTalker to encourage student to participate in commenting on activity or how they feel "That's cool" "Love That" "No Way" "I need a break" – have fun with the intonation!

- Build their own story about their day
- Contribute in circle time by answering questions using 'yes' and 'no' or any other answer set up for the classroom
- Carry messages around school or home in order to participate in social interaction such as, 'Good morning'
- Verbal support for spelling games
- Sentence scramble games